

PMOD Flipbook

Getting started with PMOD: Image Loading and Viewing

π.pmod

Getting started with PMOD: Image Loading and Viewing

Prerequisites:

- DICOM images on file system
- Demo example database

Procedure:

- Load images:
 - In DICOM format from file system
- Study information and image properties
- Save image in Interfile format

Procedure (continued):

- Append images:
 - By Autodetect from file system
 - From Demo database
- Image presentation:
 - Zoom control
 - MIP activate/deactivate
 - Color scale adjustment
 - Layout configuration
- Data inspector/SUV types
- Report creation
- Scientific capture

START PMOD v4.0 on Windows 10



E P 〇 × ダ 豆 PMOD Version 4.001	π.pmod	*				
🐉 🖛 🔂 🔻 🖬 R	Biomedical Image Quantification				and the second second	200
	Pmod Version: 4.001		Martin Cartan	8	11 12	Arr Arr
	Corporate license	*			a company a street	and a state
PXMod	Company ID: 0 License ID: 0				× ×	
Yew Yew	PMOD Technologies Ltd Employee				1 20 B	
Fuse <u>I</u> t	The License will expire in [93] days. X Close	M -jar pmod.jar			13 and	Aller .
Segment	76 Disclaimer. PMOD IS A RESEARCH TOOL. PMOD has no approval for dinical usage	nment 1.8.0_181-b13			te. Mr.	1. 19
(<u>1</u> <u>1</u> <u>1</u>	Note that the authors do not warrant performance or results of PNOD and can in no case be liable to the user of PMOD.			Application Tools	Start	– 🗆 X
Cardiac PET	Privacy Disclaimer: Patient name and ID are included in result files generated by data processing in PMOD. By experting and sharing data		File Home Share V	/iew Manage		~ 🛛
Card CT/MR	this information may be disclosed. To avoid privacy breaches it is therefore recommended using an onymized data from the outset.	Software [Version 4.001 (c)	$\leftarrow \rightarrow \cdot \uparrow$. This PC	> C_2 (D:) > Pmod4	.0 > Start ∨ č	Search Start 🔎
Flaw	lindous 10 > ord61 > 10 0		Pmod4.0	^ Name	Date modifie	d Type
	rindows 18 - 7 and 4 - 7 18.8		🧵 data	💿 RunDl	oSvr 10/30/2018	9:49 AM Windows
	lt charset: windows-1252		🧵 install	💿 RunDe	mSvr 10/30/2018	9:49 AM Windows
	e: en_US	Number of Processors 9	📜 java	RunLo	sSvr 10/30/2018	9:49 AM Windows
Neuro	i.florea -> D:\Pmod4.0 -> C:\Use	rs\i.florea		RunPr	nod 10/30/2018	9:49 AM Windows
Dop a <u>B</u> ott	nics devices: 1 [1920 x 1080], Loo	k & Feel = javax.swing.plaf.me	resources			
	Messages redirected to: D:/Pmod4.	0/system/logs/pmsys.log	3 Start			
Config	size: 21 kB [Tue Oct 30 13:58:12 C	ET 2018]	📜 system	~ <		>
X <u>Q</u> uit			4 items 1 item selected 76 b	ytes		

35 Image Processing and VOI Analysis (4.001)	- 🗆 X
DB Load © View VOIs Compare Fusion	
▶ Demo	K Reset Query & Refresh Query P 🕤
Patient Name PK* Patient ID *	
Patients (5) 🛞	Image Preview (selected "Series")
Patient name Patient ID W Modify date Sex Date of Birth PKIN1 Dyn. OFFPX bolus & MRI 2018-10-28 12.08.08.0511 M 1081.01.01 PKIN2 Dyn. DFPX bolus & MRI 2018-10-28 12.08.03.802 M 1081.01.01 PKIN3 Dyn. FDG scan with whole blood data 2018-10-28 12.07.57.053 M Image: Comparison of Comparis	
🕹 Set as "Selected for loading"	
Series [3] 💿	
Patient Name Study date Time Study description Series description Total date Last Use Mod nz nv nd nx ny Organ SN Arch User Project Diagnosis PKIN1 2006/03.01 13.02:26 CPFPX Bolus Brainmask 2015:03.01 3.2:018-10-12:12. PT 81 1 1 83 106 BRAIN 0 Pmod User1 Project Diagnosis PKIN1 2006:03.01 13:02:26 CPFPX Bolus MRAntomy 2012:0-7:271 2018:-02:69. MR 81 1 1 83 106 BRAIN 0 Pmod User1 Project Diagnosis PKIN1 2006:03.01 13:02:26 CPFPX Bolus Dmamic PET 2012:07:27 112018-10:30 10 PT 81 34 1 83 106 BRAIN 0 Pmod User1 Project Diagnosis PKIN1 2006:03.01 13:02:26 CPFPX Bolus Dmamic PET 2012:07:27 112018-10:30 10	
	Gran + () - 3 = +
Agd Add All Internet Control And All Internet Control And All Internet Add	
Selected for loading Components Administration	
21 22. 23 24 21 22. 21 22. 21 22. 23 24 24 24 24 22. 22. 22. 24 24 24 24 24 24 24 24 24 24	○ Blice ● Frame ○ All

374 Select DICOM Series	×
Series 👻	Image Preview
D:/Pmod4.0/data Grange_Eolder	
	0
	*
1.	
	@ Gay = () 3 P =
🐥 Add 🛛 🚯 Add Alf 💿 Defeter : ri@rr. Indo	m 10 💱 10
Selected for loading	0 [\$] × 100 [\$]
24	
	○ Slice ® Frame ○ All
	ACO mode (Selit by CT)
👔 🖉 with Operations 🛞	X Cancel

E Select DICOM Series Current search directory	×
Series 🐨	🗌 Image Preview:
D:/Pmod4.0/data 🐵 Change Folder 🗋 Skip atan at sta	art 🔤
I Select new DICOM folder	E I
- 2 🗟 Lookin (D:Pmod4.0/data)	0.0
Navigation button	
DATABASES dom dspcaptures	
 Double-click to navigate to: D:/Pmod4.0/data/DATABASES/Demo/data/MULTIMODALITY_PETMRI/2 	20060410/
L Sew Folder	Gray Image: Second
Select X Cancel	ACQ mode (Split by CT)
See with Operations (5)	Cancel

R Select DICOM Series	DIGONALI		×
Series 🛞	DICOM folder		Image Pregiewr
D:/Pmod4.0/data		🔲 💮 Change Folder 🔲 Skip scan a	tistan 🤤 🛞
🕮 Select new DICOM folder		×	
🝷 🗧 🗃 Look in (D:/Pmod4.0/data/DATABASES/Demo/d	ata/MULTIMODALITY_PETMRI/20060410/	- 🖻 🖪 🗞 🚭 🗙	0
Folders [1]	Hes ()		1
E Mew Folder € Select		× Cancel Rathous × Remove al Remove al	24 Carcel 24 Carcel
			× Caucei

尾 Select DICOM Series	×
Sories (2) 🐨	📄 Image Preview:
D:/Pmod4.0/data/DATABASES/Demo/data/MULTIMODALITY_PETMRI/20060410	
🗻 Patient's name Birth dale Mod Study ID SeNo w.x.h. (vol,sic) Series / Study descr. Study date Protocol No	
PFUS1 PT 80x98(35) FET / Tyrosine PET 2006.04.10 0 PFUS1 PT 80x101(35) FDG / FDG PET 2006.04.10 1	
1. Select both images by CTRL+click	
2.	
II III III III IIII IIII IIIIIIIIIIIII	
Institution Bludy Description: Tyrosine PET Series Description: FET Image Comments:	Gay - + + R D -
😽 Add 🕹 Add All 😵 Delete D 💭 🕮 Info	III 2.0 C 1.0
Selected for loading [2]	
Patient's name Birth date Mod Study ID Selvo wix h (vol,sic) Series / Study descr. Study date Protocol No	
PFUS1 PT 80x99(35) FET/Tyrosine PET 2006.04.10 0 PT 80x101(35) FDG/FDG PET 2006.04.10 1	
3.	C Slice Frame C All
Chen Contractions () North Contractions ()	ACQ mode (Split by CT)
A Catoria Concernante A Genome al 22	X Cancel



驱 Image Processing and VOI Analysis (4.001) - [PN: PFUS1, BD: , SED: FET] - [40561365]





The Image Processing and VOI Analysis (4.001) - [PN: PFUS1, BD: , SED: FET] - [40561365]



 View VOIs Compare Fusion DB Load Ê 18 (500000 07). (0.0, 100000 07) 0 PFUS1 Tyrosine PET FET [:0] 🗢 🕴 🗙 fff. 8 jus. 12 무 94 O 18 ÷ 6 × • Cold ▼ 4 b 💽 🗊 ▼ ð 2... \$ 6.544678 0.0 2 63.75 0 w 17 (500000.04) × 100 [8] (0.0, 100 16 [%] Ren. Series Information & Editing Window X · O & · 1 Å Patient / Study Into Modality Into De-Identification Info Gradient Table -Patient name : PFUS1 4DZ Patient ID : Multimodality PET & MRI . 180 x 98 08.0 m Study ID : Age, Weight, Size / Position 1, 75.0 [kg], 1.79 [m] / 37 18 (5000 (0.0, **Study Information** Patient comments * Image comments : Study description : Tyrosine PET Series description : FET . Size (x, y, z, frames) : 80, 98, 35, 1 4 Load: 📂 Database - 3 Voxel spacing (x, y, z) : 2.3438, 2.3438, 4.25 [mm] Bounding box <X>, <Y>, <Z> : <-1 172, 186.328>, <-228 516, 1 172>, <-2.125, 146.525> [mm] Tools Convert Anonym 9 180 × 72 25 Origin : -0.0, -0.0, -0.0 [mm] Units (value, time): kBg/cc, seconds 19 (50 • A Merge Frames Memory representation: [SHORT] Convert to ELOAT Ph Corrections @ Vier ? Z Solit Slices -Manufacturer / Model / Version PMOD Technologies / PMOD / 3.0 1 File name / DB ID .ITY_PET__MRI/20060410/40561365// 0 / 12 Slice Alignment 2 Edit Image History Image Properties Save: 📑 DICOM A All Origins [mm] Original Center C Defined I Hot Spot 20 [50 Z 0.0 4 Set Origins X -0.0 Y -0.0 Pixel Size [mm] -X 2.34375 Y 2.34375 Z 425 Set Pixel Size Value Units kBq/cc **V** 4 9 4 Set Units - 3 Edit Frame Times View Slice Times Ind Edit Patient / Study Info O ▶ VOIs ← View » × Close

The approximation of the second secon

affirma 1
1 P

Ŧ

 View VOIs Compare Fusion DB Load



The Image Processing and VOI Analysis (4.001) - [PN: PFUS1, BD: , SED: FET] - [40561365]



 View VOIs Compare Fusion DB Load Ê 18 (500000 07). (0.0, 100000 07) 0 PFUS1 Tyrosine PET FET [:0] 🗢 🕴 🗙 fff. 8 jus. 12 7 £ 0 18 - B × • Cold ▼ 4 b 💽 🗊 ▼ ð 2... \$ 6.544678 0.0 63.75 0 w 17 (500000.04) × 100 [8] (0.0, 100 16 [%] Ren. Series Information & Editing Window X · O & · 1 Å Patient / Study Into Modality Into De-Identification Info Gradient Table -Patient name : PFUS1 4DZ Patient ID : Multimodality PET & MRI . 180 x 98 08.0 m Study ID : Age, Weight, Size / Position 1, 75.0 [kg], 1.79 [m] / 37 18 (5000 (0.0, **Study Information** Patient comments : ÷ Image comments : Study description : Tyrosine PET Series description : FET . Size (x, y, z, frames) : 80, 98, 35, 1 4 Load: 😰 Database - 3 Voxel spacing (x, y, z) : 2.3438, 2.3438, 4.25 [mm] Bounding box <X>, <Y>, <Z> : <-1 172, 186.328>, <-228 516, 1 172>, <-2.125, 146.525> [mm] Tools Convert Anonym 9 180 × 72.25 Origin : -0.0, -0.0, -0.0 [mm] Units (value, time): kBg/cc, seconds 19 (50 • A Merge Frames Memory representation: [SHORT] Convert to ELOAT Ph Corrections ℚ Viev ? **UPZ** Split Slices w Manufacturer / Model / Version PMOD Technologies / PMOD / 3.0 1 File name / DB ID ITY PET MRI/20060410/40561365// 0 / 12 Slice Alignment 2 Edit Image History Image Properties Save: 📑 DICOM A All Origins [mm] (
 Driginal
 Center
 Defined
 Hot Spot
 20 [50 Z 0.0 4 Set Origins X -0.0 Y -0.0 Pixel Size [mm] -X 2.34375 Y 2.34375 Z 4.25 Set Pixel Size Value Units 3 kBq/cc 4 Set Units 80.75 m View Slice Times Edit Frame Times Edit Patient / Study Info O ▶ VOIs ← View » × Close

The approximation of the second secon



â

-

327

C

2





36 Image Processing and VOI Analysis (4.001) - [PN: PFUS1, BD: , SED: FET] - [40561365]



36 Image Processing and VOI Analysis (4.001) - [PN: PFUS1, BD: , SED: FET] - [40561365]

DB Load O View VOIs Compare Fusion



×





as Pmod (ope	en): AUTODETECT: Select hea	Home directory				×
💌 🤤 🗃 Look In	D:/Pmod4.0/data/			• 🖻 🖆	1 🌬 🗲) ×
Folders [5]		Files	Image Preview			
DATABASES	M4					
dicorn dispcaptures						F
	NE					-4
		1 Double-click to pavidate to:				-
	D:/Pmod4	.0/data/DATABASES/Demo/data/MOUSE_CTSPECT/2007050	9/			
						-
			□ = = <u>=</u>	- ×	•	•
		🖟 Add to Selected 🐳 Add all to Selected 🗸 Selectali 🗴 🕲 Delete		1.11	_	
Coloring City	416		10 1 1	10		1
Selected Files			Gray	= 4	- 10 -	×.
		24	an [0.0		1.0	
			0 [8]	Q	100	[\$]
			E) Di v	-	BIR	Ŧ
			Slice () Fra	ne 🔾 All		
	Middan)	😥 with Operations 🛞 🗹 Remove after loading		× Cancel		





The Image Processing and VOI Analysis (4.001) - [PN: P3D1, BD: , SED: SPECT] - [06350200]

DB Load O View VOIs Compare Fusion



A View » 🕼 🖞 🕎 💬 🚱 🏗 🔺 🗹 Synchronize inyout 🖉 Beport 3 🔺 🔻 🔅 ∓ 🛓 Loaded

Ē

-

Image Processing and VOI Analysis (4.001) - [PN: P3D1, BD: , SED: SPECT] - [06350200]	- a ×
E Select Serie(s) 1. Database selection	×
Demo	SQ 🗙 Reset Query 🔅 Refresh Query 🧟 😁
Patient Name PET" 2. List names starting with "PET" + Enter-key Uodified Uodified	
Patients [2] 🛞	Image Proview (selected "Series")
Patient name Patient ID To Modify date Sex Date of Birth	
PETCT Example Whole Body Tumor 2018-10-26 12:06:50.379 M PET TEMPLATE PET 2018-10-26 12:06:47.223	
4.	6 ÷
Set as "Selected for loading" 🕼 Edi Patient 😵 Delete Patient(s) 📽 Create new Patient Set Project 😂 Marga 🛁	Split
Series (2) 🕲	-
Patient Name Study date Time Study description Series description Time Modified Last Use Mod nz nv nd nx nv Organ SN Arch User Project Diagnosis PETOT Example 2009.00.05 10.01.23 PDG PET CT 2010-09-03 10. 2018-10-24 08. CT 313 1 1 512 S12 Pmod user 1 * PETOT Example 2009.00.05 10.01.23 FDG PET PET 3D 2010-09-03 10. 2018-10-25 09. PT 313 1 1 128 128 Pmod user 1 * Study description Time Modified Last Use Mod nz nv nd nx ny Organ SN Arch User Project Diagnosis PETOT Example 2009.00.05 10.01.23 FDG PET PET 3D 2010-09-03 10. 2018-10-25 09. PT 313 1 1 128 128 Pmod user 1 1 Study description Study description Study description D	
6.	
Add All Into Edit 😒 Delete 🐰 Set Project. 🔻 🕬 🖉 🖉 🗸 📲	
Selected for loading [1] Components Administration [7]	H 0.0 1 0 1.0
Patient Name Study date Time Study description Senes description Modified Last Use Mod nz nv nd nx ny Organ SN Arch User Project Diagno PETCT Example 2009,06.05 10:01:23 FDG PET CT 2010-09-03 16. 2018-10-24 08 CT 313 1 1 512 Demo user 1 4 7. <td>24 0. [8] 100 [8] 0. Slice ® Frame O All 41</td>	24 0. [8] 100 [8] 0. Slice ® Frame O All 41
Deen 😹 with Operations 🕐 📾 🕸 🖾 Kernove 🛪 Bernove all 🖉 Remove all 🖉 Remove all 🖉 Remove all 🖉 Remove all 🖉 CT ACQ 📄 🔺 📢	* <u>C</u> ancel



DB Load O View VOIs Compare Fusion





🔺 View » 🔹 Ol 📅 🏧 😥 🚱 🗈 🔺 🗹 Synchronize Inyolit 🔄 Beport 3. 🔺 🛡 🚳 ∓ 🛓 Loaded

The Image Processing and VOI Analysis (4.001) - [PN: PETCT^Example, BD: , SED: CT]







A View » 🕼 🖞 🕎 💬 🚱 🖾 🔺 🖉 Synchronize Involut 🖉 Beport 3 🔺 🛡 🐼 😤 🛓 Loaded



DB Load O View VOIs Compare Fusion

- 🗆 X





- View ... 🗱 🖬 📰 📰 😨 😨 🙃 🔺 🗹 Synchronize layout 🗟 Becott 3 🔺 🛡 🛓 🕇 🛓 Loaded

3 Image Processing and VOI Analysis (4.001) - [PN: PETCT^Example, BD: , SED: CT]

×

35 Image Processing and VOI Analysis (4.001) - [PN: PETCT^Example, BD: , SED: CT]



DB Load O View VOIs Compare Fusion



DB Load O View VOIs Compare Fusion



DB Load View VOIs Compare Fusion



35 Image Processing and VOI Analysis (4.001) - [PN: PETCT^Example, BD: , SED: CT]



Blad Verw Vols Compare Fesion	\times			D: CT]	PETCT^Example, BD: , SI	s (4.001) - [PN:	nd VOI Analysis	🛚 Image Processing a
Image Project Image Project Patient Name Point Patient ID Modified Patient Name Point Concept Nature & Modify date Privint Dyn. CPEPX bolus & MRI 2016-10-26 12/08/08/07 M PRIVIN Dyn. DASB without blood date 2018-10-26 12/07/26/02 M Privind Dyn. PASE without blood date 2018-10-26 12/07/26/02 M Modified 2018-10-26 12/07/26/02 Modified 2018-10-26 12/07/26/02 Modified 2018-10-26 12/07/26/02 Modified Sea "Selected for loading" Set as "Selected for loading" Set Projed Merge © Selft			Database selection	1		sion	Compare Fu	DB Load O View VOIs
Image Project Image Project Image Project Image Project Patient Name Patient Io Image Project Image Project Image Project Patient Io Image Project Image Project Image Project Image Project Patient Io Image Project Image Project Image Project Image Project Patient Io Image Project Image Project Image Project Image Project Patient Io Image Project Image Project Image Project Image Project Patient Io Image Project Image Project Image Project Image Project Patient Io Image Project Image Project Image Project Image Project Patient Ion Image Project Image Project Image Project Image Project Patient Ion Image Project Image Project Image Project Image Project Image Project Patient Ion Image Project <	\times	\sim	Batabace selection					∞ Select Serie(s)
Patient Name KIN* 2. List names starting with "PKIN" + Enter-key Birth Date: Image Preview (selected Select	uery 🤗 🕤	🔽 4 🕨 🗆 Search All 🐐 New 🛛 SQ 🛛 🗙 Reset Query 😂 Refresh Query						▶ Demo
Patients [5] € Image Pregiew (selected "State of the sex patient is the sex pati		Birth Date:	with "PKIN" + Enter-	mes starting	2. List na			Patient Name PKIN* Patient ID *
Patient name Patient D Modify date Sex Date of Birth PKIN2 Dyn. CPFPX bolus & MRI 2018-10-26 12:08:06.161 M 1981.01.01 M 1981.01.01 PKIN2 Dyn. FDG scan with whole blood data 2018-10-26 12:07:52.757 PKIN4 Dyn H2D brain scan, Baseline 2018-10-26 12:07:49.664 2018-10-26 12:07:49.664 Carter with Metabolites, KM File only 2018-10-26 12:07:49.664 Set as "Selected for loading" Image: Selected for loading" Edit Patient Selected for loading Marge < Split Set as "Selected for loading" Set as "Selected for loading"	3S")	Image Preview (selected "Series")						Patients [5] 💿
↓ Set as "Selected for loading" Ic∞ Edit Patient S Delete Patient(s) * Create new Patient Set Project ⇒ Merge ⇒ Split		tent	3. Select pa	▼ Modify date 2018-10-26 12:08:06.161 2018-10-26 12:08:03.802 2018-10-26 12:07:57.053 2018-10-26 12:07:57.053 2018-10-26 12:07:57.657 2018-10-26 12:07:49.664	ent ID CPFPX bolus & MRI DASB without blood & MRI DASB without blood data 420 brain scan, Baseline 420 brain scan, Baseline rr with Metabolites, KIN File only	Patie Dyn. C Dyn. C Dyn F Dyn H Trace		Patient name PKIN1 PKIN2 PKIN3 PKIN4 PKIN5 4.
		ate new Patient Set Project B⇒ Merge =\$ Split	I™ Edit Patient 😮 Delete Patient(s) 📽 C				cted for loading"	Set as "Selv
				- warren brennen buen			atolo Prince	Series [3] 💌
61 2006/03.21 15:15:07 DASB SERT Brainmask 2016-10-81 fb. 2018-10-31 08 PT 32 1 1 97 BRAIN Pmod User1 * 2006/03.21 15:15:07 DASB SERT Manage 2011-09-2111 2018-10-31 08 PT 32 1 1 59 79 BRAIN Pmod User1 * 2006/03.21 15:15:07 DASB SERT MR Anatomy 2011-09-2111 2018-10-31 08 PT 32 33 1 59 79 BRAIN Pmod User1 * 2006/03.21 15:15:07 DASB SERT Dynamic DASB PET 2010-09-03 16 2018-10-31 08 PT 32 33 1 59 79 BRAIN Pmod user1 *		Pmod User1 * Pmod user1 * Pmod user1 *	112 110 111 111 Organi SN 32 1 1 59 79 BRAIN 0 124 1 1 137 178 BRAIN 0 32 33 1 59 79 BRAIN	wooline Last Ose Mod 2016-10-06 16 2018-10-19 12 PT 2011-09-21 11 2018-10-31 08 MR 2010-09-03 16 2018-10-31 08 PT	Brainmask MR Anatomy Dynamic DASB PET	DASB SERT DASB SERT DASB SERT	Male Hitle 03.21 15:15:07 03.21 15:15:09 03.21 15:15:07	6. 2006 2006 2006
5. Select series	•	S	5. Select serie	m				
Add All Ice Edit S Delete			I🕶 Edit 😮 Delete 幾 Set Project				🐥 Add All	₽ A <u>d</u> d
Selected for loading [1] Components Administration [23]	-					stration [23]	Components Adminis	Selected for loading [1]
Patient Name Study date Time Study date Study date Stud	100 [\$]	Arch User Project Diagnost Demo user 1 *	nz nv nd nx ny Organ SN 32 33 1 59 79 BRAIN	Modified Last Use Mod 2010-09-03 16 2018-10-31 08 PT	Series description Dynamic DASB PET	Study description DASB SERT	date Time 03.21 15:15:07	Patient Name Stud PKIN2 2006
Image: Contraction in the second	4	move after loading CT ACQ All	Server all ∠ Remove all .	W		۲	🕫 with Operations	Pen 2
		X Çancel						


▲ View 🗤 🐲 OI 🗄 🏧 😨 😨 🛣 🔺 🗹 Sinchronize layout 🔿 Becort 3 🔺 🛡 🚔 🕇 🛓 Loaded









- View » 🕼 OI 🖞 🏧 🚱 🏗 🔺 🗹 Synchronize Inyout 🖉 Beport 3 🔺 🛡 🚳 🐔 🗜 Loaded



A C PKIN2 [DA3B PET] DA5B SERT. • • • • × State Inspector [Dynamic DASB PET] x(33) = 128.0[mm], y(42) = 125.5[mm], z(16) + Value: 56.73117, Time: 2250.0 (2100.0: 240) F Distance: 0.0 [cm] TAC Profiles 3D plot & histogram SUV coll Settings Scale Time Vector To Global MinMax
x(33) = 128.0[mm], y(42) = 125.5[mm], z(16) L Value: 56.73117, Time: 2250.0 (2100.0: 240) P Distance: 0.0 [cm] TAC Profiles 3D plot & histogram 3UV vol Settings □ Scale Time Vector To Global MinMax
Scale Imme Vector I o Global Jummiax
-0.3 23.7 47.1 70.6 Send to 117.5 Sices -0.3 -0.3 -0.3 -0.3 -0.3 -0.3 Sices -0.3 -0.3 -0.3 -0.3 -0.3 -0.3 -0.3 -0.3 Sices -0.3
R DL C Imme vector

▲ View » 🕼 🖞 🏧 🙀 🖾 🔺 🖉 Synchronne inyout 🖉 Beport 3 🔺 🛡 🐼 😤 🛓 Loaded

nage Processing and VOI Analysis (4.001) - [PN: PKIN2, BD: , SED: Dynamic DASB PET]

Ē

 \times

-

Image: A control of the lange of the la	ad O View VOIs Compare Fusion				
<pre></pre>	A	A 🗷 🗷 Data Inspe	ector [Dynamic DASB PET]	О РКІМ2 [D	ASB PET]DASB SERT▼ ())
Image: State of the		x(33) = 128.0[mm ★+ Value: 56.73117 Pi Distance: 0.0 [cm TaC Probles	n), y(42) = 126.5[mm], z(16) = 68.0[mm] , Time 2250.0 (2100 0 : 2400.0) [seconds] n] 3D ni/of & histogram Stuy VOI Settings	View probe Unit kBg/cc	▼ 4 ▶ [3] ▲ ▼ ♥ 4 ▶ [3] ▲ ▼ ♥ 90.0
H Duré de la danie da da danie da danie da	R R	Requested information for SUV calculation	Patient weight [%g] 75.0 Patient height [m] 1.79 Patient height [m] 1.79 Patient sec M ▼ Date / Time (of decay correction) (○ Series @ Acquisition ○ Scan) Image decay correction Date / Time 1 1 1 / 0 0 0 0 Radionucide half-life [seq] 1223.4 11 C (20.39 i Radionucide half-life [seq] 120.0 0 0 0 0 0 0 0 0 0 0 0 0 0 0 0 0 0 0	000000 R≥ 000000 R≥ n) v + > v + >	- Of -
R Copy to Study Into Image is not marked as attenuation corrected. Image decay correction time is missing. Selected Date / Time will be used instead. ? Close	P H	SUV Types	Administration Date / Time 2006 3 21 15 15 7 Dose remaining in syringe [MBq] 0.0 8 8 7 15 15 7 15 15 15 7 15	method) y Janmahasatian method) Jalis method)	t Anonym cnes v lices v
	8 B L	1.	Copy to Study Into Image is not marked as attenuation corrected. Image is not marked as decay corrected. Image decay correction time is missing. Selected Date / Time will be used Cose a	instead. ?	AI
	41 4 16 P 18 18 4 4 10 a × 4				▶ <u>V</u> OIs

nage Processing and VOI Analysis (4.001) - [PN: PKIN2, BD: , SED: Dynamic DASB PET]

Ka Ka I

Ē - \times 馬 Image Processing and VOI Analysis (4.001) - [PN: PKIN2, BD: , SED: Dynamic DASB PET]

























4

0

9

×

2.

w.

86A

•

• ?

-

🔺 🗌 All

0

9 O

•

ð

8

ŧ

O I

● B & ⊵ Ø B

🗄 🖄 3D 🔭

Load: Patabase V

Save: 📑 INTERFILE

▶ <u>V</u>OIs

Merge Frames

1. Sagittal layout, radiological

€ U N 44 4 33 9 19 1 € € 10 ▲ × €







35 Image Processing and VOI Analysis (4.001) - [PN: PKIN2, BD: , SED: Dynamic DASB PET]

D,

X









區 Image Processing and VOI Analysis (4.001) - [PN: PKIN2, BD: , SED: Dynamic DASB PET]

















â 4PZ ?
























🔺 View » 🔹 Ol 🛐 🚎 🚱 🕰 🔺 🖉 Synchronize Isyout 🔗 Beport 3 🔺 🛡 🛎 🏋 🛓 Loaded

(C)





